

Please watch our video at www.retainersdirect.com before you take your impressions.

User instructions:

Step 1: Select the impression tray that fits your mouth best. Stand in front of the mirror.

Step 2: Open one set of putty, one base and one catalyst.

Ensure that your hands are clean to prevent dust or dirt from contaminating the putty.

Step 3: Use fingers to knead the impression material to a uniform color.

It is best to do this within 45 seconds or less.

Step 4: Roll into the shape of a log. Lay the log in the upper tray in a U shape that mimics the shape of your teeth. Do not smash down. The log should go all the way to the back of the tray.

It is best to do this within 90 seconds or less.

Step 5: Place the tray in the mouth so your teeth are positioned over the center of the putty log.

Press the tray straight up until the tray stops with the teeth going into the center of the putty log.

Hold the tray in place with your thumbs for 3 minutes without moving the tray.

Step 6: Open the mouth and pull the tray straight down to remove.

Step 7: Repeat the procedure with the properly fitting lower tray. You will get a perfect impression.

Don't worry if the first impression is not perfect. In your kit is an extra set of putty to try again.

Cautions:

Do not use rubber gloves when you mix the putty.

The working time and setting time will be affected by temperature.

Storage:

Protect against moisture and keep in a dry place.



245*198mm